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PROLUBE®

Augentropfen

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INSTRUCTION FOR USE

PROLUBE® Eye Drops

For moistening and lubrication of the eye in case of irritation, burning or itching sensations, foreign item sensation, feeling of dryness or use of contact lenses.

WHAT ARE PROLUBE® EYE DROPS?

Prolube® Eye Drops moisten the eye, a pleasant, velvety smooth film provides immediate relief.

WHAT DO PROLUBE® EYE DROPS CONTAIN?

Prolube® Eye Drops contain 4.3 mg/ml methylhydroxypropylcellulose (Hypromellose), glucose, glycine, propylene glycol, sodium chloride, potassium chloride, magnesium chloride, calcium chloride, 0.05 mg/ml benzalkonium chloride, purified water.

WHAT ARE THE PROPERTIES OF PROLUBE® EYE DROPS?

Prolube® Eye Drops are an aqueous, hypotonic and slightly viscous solution, its unique composition provides best comfort.

Prolube® Eye Drops contain a unique combination of high quality ingredients, ensuring excellent tolerance and comfort by sustained moistening. The nutritive components glucose and glycine as well as the physiological tear-ions are responsible for recovering of the natural balance of the eye.

Prolube® Eye Drops do not leave any unpleasant residues.

THERAPEUTIC INDICATIONS – WHAT ARE PROLUBE® EYE DROPS USED FOR?

Prolube® Eye Drops help to moisten and lubricate the eye in case of burning or itching sensations, irritations, foreign body sensations or feeling of dryness. These symptoms may be derived by one or more of the following conditions:

Use of contact lenses, dry air, pollution like dust or smoke, wind or draft, prolonged stay in heated or air-conditioned rooms, prolonged screen work, climate change (e.g. on travel), prolonged stay in transport vehicles (car, train, plane),...

DOSAGE AND METHOD OF ADMINISTRATION

HOW TO USE PROLUBE® EYE DROPS

Several times a day (up to 6 times per day), depending on the individual requirements, 2-3 drops are instilled into the conjunctival sac of the eye.

To avoid contamination, do not let the dropper touch your eye or anything else. Put the cap back on and close the bottle straight after you have used it.

Prolube® Eye Drops can be used over a longer period of time (up to 30 days).

WHEN SHOULD PROLUBE® EYE DROPS NOT BE USED?

You should not use **Prolube® Eye Drops** if you are allergic to any of the ingredients of the product.

POSSIBLE SIDE EFFECTS OF PROLUBE® EYE DROPS/INTERACTIONS WITH OTHER PRODUCTS

There are currently no undesirable side effects known.

No interactions with medicinal products are known. Simultaneous use with ophthalmic preparations should anyway *follow the advice of the prescribing physician*.

PRECAUTIONS FOR SAFE USE

Prolube® Eye Drops contain the preservative benzalkonium chloride which may cause eye irritation. Benzalkonium chloride is known to discolour soft contact lenses. Avoid contact with soft contact lenses. Remove contact lenses prior to application of **Prolube® Eye Drops** and wait at least 15 minutes before reinsertion.

You must not use the bottle if the tamper-proof seal on the bottle neck is broken before you first use it.

Prolube® Eye Drops are not a medicinal product! **In case of persisting symptoms, you should consult a physician.**

Keep out of the reach of children!

PACK SIZE: 10 ml

SHELF LIFE

See expiry date on the label and package.

Do not use longer than 4 weeks after first opening.

STORAGE INSTRUCTIONS

 ^{25°C} Do not store above 25 °C.

 **Manufacturer:** MoNo chem-pharm Produkte GmbH,
Leystraße 129, 1200 Vienna, Austria

Distribution: SIGMAPHARM Arzneimittel GmbH, A-1200 Vienna

 0123

 **LOT** Batch Number

 Expiry date

 **STERILE A** Sterilisation using aseptic techniques

 Take notice of instructions for use!

Revision date: April 2019