Vliwasorb® adhesive

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en Instructions for Use

Product description

Vliwasorb® adhesive is a flexible and soft, superabsorbent, self-adhesive wound dressing used for wounds with moderate to high amounts of exudate. Vliwasorb® adhesive consists of a superabsorbent polymer (sodium polyacrylate) which can bind high quantities of wound exudate, imbedded in cellulose, and a polyurethane support layer with an acrylate adhesive.

The wound dressing binds excess exudate and thereby protects the wound and wound environment against maceration.
The smooth, elastic polyethylene wound contact layer reduces the risk of adhesion to the wound.*

Product composition

Polyethylene, polypropylene, cellulose, polyethylenterephthalate, ethylene vinyl acetate copolymer, green tea extract, polycrylate, polyurethane

Intended use

Absorbent multilayer wound dressing indicated for exudate management of acute and chronic wounds.
The size of dressing depends on the wound size. The dressing should be placed in the middle of the wound and should overlap the wound by approx. 2-3 cm. Smooth the dressing gently over the wound and press down at the edges.
In case of deep wounds, an appropriate wound filler may be used, depending on the wound status. For treatment of underlying diseases Rosidal® compression bandages can be used.

Indications

Vliwasorb® adhesive is indicated for the treatment of wounds with moderate to high amounts of exudate:
• pressure ulcers, arterial ulcers, venous leg ulcers, diabetic ulcers
• postoperative wounds healing by secondary intention
• closed laparotomy wounds
• superficial 2nd degree burns
It can also be used as a temporary dressing with an appropriate wound filler (recommended are Suprasorb® Liqueal Hydroactive Fibr Dressing, Suprasorb® A Calcium Alginate Dressing, Suprasorb® A + Ag Antimicrobial Calcium Alginate Wound Dressing or Opraclean® gauze Dressing with iodoform) for treating deep wounds.

Contraindications

Known allergy and / or hypersensitivity to any of the product components.
Do not use in tunnelling wound pockets as the product can expand considerably when wound exudate is absorbed. Do not use on wounds with light exudation, as the wound may dry out and suction pain may occur.

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Side effects

In isolated cases skin irritations and / or allergies may occur.

Warnings and precautions

Do not use Vliwasorb® adhesive if the package is damaged or accidentally opened. Do not cut or tear Vliwasorb® adhesive.

Application

Application is performed by healthcare professionals. Prior to use, visually inspect the sterile barrier system for damage to the packaging integrity. Open the packaging at the designated place and remove the product in a sterile manner.

1. Preparation of the wound:

Cleanse the wound as usual prior to applying the new dressing (e.g. with sterile saline solution, Ringer's solution) and pat the wound edges dry.

2. Application of the wound dressing: The size of dressing depends on the wound size.

The dressing should be placed in the middle of the wound and should overlap the wound by approx. 2-3 cm. Smooth the dressing gently over the wound and press down at the edges.
In case of deep wounds, an appropriate wound filler may be used, depending on the wound status. For treatment of underlying diseases Rosidal® compression bandages can be used.

3. Dressing change: The healthcare professional will determine the dressing change intervals for Vliwasorb® adhesive, depending on the wound status and degree of exudate from the wound.

Disposal

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General instructions

In case of serious incident, contact the manufacturer and the responsible health authorities.

* The stated properties are based on in vitro data.

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