DE: BioBloom CBD pain patch Gebrauchsinformation

vers. 1.1/ February 2022

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UK: BioBloom CBD pain patch instructions for use

Please read instructions for use carefully before using the BioBloom CBD pain patch.

How does BioBloom CBD pain patch work?

BioBloom CBD pain patch is a topical product with acrylate copolymer solution, which helps relieve muscle/joint pain.

BioBloom CBD pain patch is a medical device. Efficacy and safety have been tested according to the European Directive for Medical Devices (93/42/EEC).

Active Principle of BioBloom CBD pain patch

BioBloom CBD pain patch is used for the relief of muscle/joint pain (such as arthritis, back pain, sprains, sports injuries). **BioBloom CBD pain patch** provides a long lasting pain relief.

Application areas

For relief of pain and swelling (such as arthritis, back pain, sprains, sports injuries).

Dosage instructions, type and duration of application

BioBloom CBD pain patch should only be used on the skin. Clean and dry the affected area before applying the product on the skin. Remove the cover from the patch and apply it to the affected area. Do not apply the patch to injured or irritated skin (e.g. scratched, sunburned, infected or on skin with cuts or rash). The patch can be used once a day. Follow all instructions on the product packaging.

Remove the patch before showering and do not apply it immediately after shower. Do not apply bandages or do not wrap the area where the patch is applied. Do not cover the treated skin area with a heating pad as this may increase the risk for side effects.

If other products are applied to the skin area of the patch - wash it well off and wait around 2 hours before using the patch. Wash your hands well after handling the patch. Avoid getting the patch in your eyes, nose or mouth. After the patch removal, fold the sticky sides of the used patch and dispose of it in the household waste.

The product is for temporary use.

If there is no improvement 2 – 3 days after starting the treatment or if the symptoms have worsened considerably, please consult a doctor.

General precautions

- for external use only
- do not use this product on damaged or injured skin
- avoid eye contact. Wash hands carefully after use

- not recommended for children and adolescents under 14 years of age. Use only after consulting your doctor
- keep out of reach of small children
- keep out of reach of pets
- persons with an allergy to alcohol, solvents or any of the ingredients should not use the product
- during pregnancy or breastfeeding, use only after consulting a doctor

Contraindications

BioBloom CBD pain patch must not be used in cases of known hypersensitivity to any of the ingredients.

Interactions

To avoid drug interactions, you should apply medication 2 hours before or after use or consult your doctor.

Incident reporting

If this product is associated with a serious incident (death of a patient, user or other person; temporary or permanent serious deterioration of the health status of a patient, user or other person; serious risk to public health), this incident must be reported to the manufacturer and the National competent authorities.

Instructions and information on the shelf life of the medical device

BioBloom CBD pain patch may only be used until the date marked “usable until” on the folding box. Due to the nature of the ingredients, a slight color change may occur after some time using the product. This does not affect the quality of the product.

Dosage form and content

Original packaging with 12 patches.

Composition

Active ingredients: Acrylate Copolymer, Ethyl Acetate, Ethanol, Heptane, Pentanedione. Other ingredients: Cannabidiol (CBD), Arnica Tincture, Menthol. Contains alcohol. Without preservatives and colourings.

Storage

Keep cool (between 10°C and 25°C), dry and protected from sunlight.

Disposal

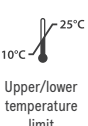
The product can be disposed of with normal household waste.

Manufacturer:
CANNORDIC A/S
Fruebjergvej 3
Copenhagen 2100
Denmark

Symbol CE

Upper/lower
temperature
limit

Use
by

Please
observe the
instructions
for use

Batch
number

Protect
from
sunlight

Do not use
if the
packaging
is damaged

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Distribution: BioBloom GmbH, Frauenkirchenstr. 12, 7143 Apetlon, Autriche, office@biobloom.at, www.biobloom.com