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EN: Instructions for use – BUDDYCARE®MED Kine-Tape. Product description: Elastic physio tape for support in cases of muscle, joint, and ligament pain. **Designation & intended purpose:** Name: BUDDYCARE®MED Kine-Tape. **Type:** Non-invasive, non-active, single-use tape (non-sterile). **Application:** Supports muscle function, relieves pain, stabilizes joints, and promotes microcirculation and lymphatic flow through mechanical stimulation via the skin. **Target group:** Adults and adolescents aged 12 and over. **Users:** Professionals (e.g., therapists) and trained laypersons. Trained laypersons are people who have been instructed in the correct use by medical professionals (e.g., physiotherapists, doctors). Training is provided, for example, through personal instruction. **Clinical benefit:** The use of the tape is intended to relieve pain, improve joint stability, and support natural healing through mechanical stimulation. **Features:** Water-repellent, breathable, elastic (up to 140%), skin-friendly, adheres for several days without loss of effectiveness. **Indications:** Muscle tension/hardening, joint pain, instability, tendon/ligament irritation, lymphatic congestion (e.g., after sports injuries). **Contraindications:** Do not use in cases of: Open/broken skin, skin conditions (e.g., neurodermatitis, psoriasis), allergies to acrylic adhesive/cotton, thrombosis, circulatory disorders, unexplained pain without medical consultation. **Warnings and precautions:** For external use only on healthy, intact skin. Product is not sterile – do not apply to wounds. Single-use product – do not wash or reuse. Discontinue use immediately if skin irritation (itching, redness, swelling) occurs. Do not use if the packaging is damaged or if the expiration date has expired. Keep out of reach of children. Do not apply too tightly to avoid impaired circulation or skin irritation. If symptoms persist, seek medical advice; reuse may lead to skin irritation, loss of effectiveness, or hygiene problems. Serious incidents related to the product must be reported to the manufacturer (info@bugslack.at) and the relevant national authority. **Skin preparation:** Cleanse skin, keep it dry and free of grease. Remove hair if necessary. Test compatibility on an inconspicuous area (e.g., forearm). **Application:** Cut the tape to the required length, round off the corners. Tear the protective film in the middle and remove. Apply with gentle tension. Always attach the ends without pulling. Rub gently with the palm of your hand to activate adhesion. **Wear time:** up to 5 days (showering is possible, do not bathe). **Material:** 95% cotton, 5% elastane. Acrylic adhesive (latex-free). Not a medicinal product / no wound care. **Storage:** Cool, dry, protected from light (5–25°C). Protect from sunlight and moisture. Keep out of reach of children. **Disposal:** Dispose of with household waste; observe regional regulations. Further information: Application videos and multilingual instructions available at: www.buddycare-med.com. Status: June 2025 – Version 1.0. This document meets the requirements of EU Regulation 2017/745 (MDR), Annex I, Chapter III, Section 23.4.

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