
Suprasorb® P SensiFlex border, multisite border

Silikon-Schaumverband
Pansement hydrocollulaire silicone
Silicone Foam Dressing
Apósito de espuma y silicona
Ligadura esponjosa de silicone
Medicazione in schiuma di silicone
Siliconen schuimverband
Silikone-skumforbinding
Silikonskumförband
Silikon-skumbandasje
Silikonivaahattosidos
Silikonový pěnový obvaz
Silikonový pěnový obváz
Szilikon habkötés
Opatrunek z pianki z warstwą silikonu
Obloga iz silikonske pene
Povoj sa silikonskom pjenoj
Απορροές ολικωοπονο επίθεμα
Silikonlu köpük yara örtüsü

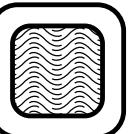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en Instructions for use

Product description

Suprasorb® P SensiFlex (multisite) border is composed of:

- a silicone wound contact layer,
- a flexible wound pad consisting of three layers: an absorbent polyurethane foam layer, a non-tissue distribution layer and a super-absorbent core; the distribution layer and the absorbent core are perforated, which makes the wound dressing more flexible,
- a vapour-permeable and waterproof protective backing.
- The five-layer construction facilitates dynamic fluid management to provide and maintain optimal moist wound environment which leads to the promotion of faster wound closure and may help to reduce the risk of maceration. The gentle silicone layer can be lifted and repositioned without losing its adherence. Also, this layer can minimise pain to patients and trauma to wounds and the surrounding skin during dressing changes. The absorbent padding layer provides superior absorption and locks exudates away.

Product composition

Silicone, polyurethane, polyacrylate, fluff pulp and polyolefin

Intended purpose

The Suprasorb® P